



# **Snelle regeneratie**

# **Herstelde beweging**

Toepassingsgids





/vPovv uvPPov ov DuuoDvuul vPu  
 vPuvPoovvoPlvuvPvuvvvvuooilv  
 vPvvdvlvldPoilivP

/vvDuuoD vvuooill uvduvoilv  
 vvvvv vrsOPvPvv vroV  
 vW

o vPwoUwvPUtv U wPv DuuoDv  
 tv be e e e e e ee e em e me  
 e e e e e e e e e e e e  
 o e e m e m e e e eb ee e e  
 m e ebe e b me

- Place STSG as soon as granulation tissue has grown through the matrix:
- o If applicable **remove potential debris and/or hypergranulation** from the wound surface before grafting.
  - o Place STSG as described in step 3 (Application of STSG) and apply and repeat step 4 (Wound Coverage of STSG) and step 5 (Dressing Changes and Aftercare) as described before.

## Appearance of STSG at 1<sup>st</sup> Dressing Change

Wound appearance may not be as pink as with STSG only (Ghosting Effect). The suspicion could be that the graft did not take. However, at day 5, vascularization starts but MatriDerm® Dermal Matrix has not been fully converted to dermal tissue yet. Therefore, a pale appearance of the wound at the first dressing change might occur but healing proceeds normally. Re-assess the wound at the next dressing change.



Example: day 5 post-operative (Ghosting Effect)



Example: day 14 post-operative (95% take rate)



MedSkin Solutions Dr. Suwelack AG  
 Josef-Suwelack-Strasse 2  
 48727 Billerbeck | Germany

P +49 (0)2543 2182-0  
 E info@medskin-suwelack.com  
 W www.medskin-suwelack.com

1. Panayi A. et al. Evidence based review of negative pressure wound therapy, World J Dermatol. Feb 2, 2017; 6(1): 1-16  
 These recommendations are designed to serve only as a general guideline, and are not intended to supersede institutional protocols or professional clinical judgment concerning patient care. Please also consult MatriDerm® Dermal Matrix instruction for use.  
 All product and company names are trademarks™ or registered® trademarks of their respective holders. Use of them does not imply any affiliation with or endorsement by them.